

WHY GBT?

1. A GAME CHANGER FOR BETTER ORAL PROPHYLAXIS
2. GOOD ORAL HEALTH = HEALTHY PATIENT
3. BEST CLINICAL PROTOCOL = BEST CLINICAL RESULTS. GBT REDUCES THE RISK OF CARIES, GINGIVITIS AND PERIODONTAL DISEASE.

“We can be sure only when biofilm has been completely removed.”

Nicole Einemann and Manuela Cordes, Dental Hygienists, Dr. Arendt Dental, Bremerhaven, Germany

4. PATIENTS WANT GBT. NOTHING ELSE. NO MORE FEAR OF PAIN. NO MORE DISCOMFORT. ONCE EXPERIENCED, PATIENTS LOVE GBT, ESPECIALLY CHILDREN.
5. SATISFIED CLINICIANS, RELAXED PATIENTS. EFFICIENT, SYSTEMATIC WORKFLOW WITH ERGONOMIC INSTRUMENTS. BYE-BYE STRESS, BYE-BYE CARPAL TUNNEL S., BYE-BYE BIOFILM.
6. SUCCESSFUL DENTAL PRACTICES - GOOD MONEY FOR GOOD WORK. ENTHUSIASTIC PATIENTS WILL RECOMMEND THE GBT-CERTIFIED PRACTICE.
7. ORAL MEDICINE & PREVENTION ARE THE FUTURE OF DENTISTRY. GBT IS THE FUTURE OF PREVENTION.
8. GBT IS A SURPRISING “I FEEL GOOD EXPERIENCE”. STATE-OF-THE-ART IS FAVORED BY PATIENTS. AND BY CLINICIANS.
9. REGULAR GBT RECALL GUARANTEES LASTING ORAL HEALTH FOR YOUR PATIENTS.



More about GBT



Ems-dental.com



EMS 
MAKE ME SMILE.

Reserve a Live GBT Demo: gbt-dental.com/GBT-LIVE-DEMO-ENG

HOW TO DO GBT?

01 ASSESSMENT AND INFECTION CONTROL

Protect your patients and your practice team from infections. Start GBT by rinsing 40 seconds with BACTERX® Pro. Diagnosis and documentation of the individual supra- and subgingival status for each patient.

02 DISCLOSE

Make the oral biofilm visible by disclosing and explain the **WHY** to the patient. Scientific evidence confirms that oral biofilm causes 95% of all dental problems: caries, gingivitis and periodontitis.

03 MOTIVATE

Emphasize the importance of oral home care and professional prevention. Instruct your patients in oral hygiene.

04 AIRFLOW® MAX

Eliminate the disclosed biofilm supra- and subgingivally up to 4mm with AIRFLOW® MAX and AIRFLOW® PLUS Powder. The original AIR-FLOWING® is minimally invasive and pain-free.*

05 PERIOFLOW®

Remove oral biofilm in deep pockets up to 9mm with PERIOFLOW® Nozzle and AIRFLOW® PLUS Powder.*

06 PIEZON® PS / PI MAX

Gently remove calculus on natural teeth with the original PIEZON® NO PAIN PS (Perio Slim) Instrument and around implants, restorations and orthodontic appliances with the new PIEZON® PI (Peri Implant) MAX Instrument.*

07 CHECK

Check whether oral biofilm and calculus have been removed completely and whether previously hidden caries is now visible. Conventional professional teeth cleaning removes only 50% of oral biofilm in areas that are difficult to reach.

Source: Stiftung Warentest 7/2015. **NO MORE POLISHING.** Polishing and grinding with abrasive pastes is truly no longer necessary. Why produce unnecessary smear layers?

08 RECALL

Would your patient like to answer the GBT questionnaire?
This is helpful for a successful GBT treatment.
Schedule the next prophylaxis appointment.

*Learn more about how GBT works in practice, how to avoid dust when AIR-FLOWING®, how to use PERIOFLOW® correctly or how to minimize the risk of emphysema at the Swiss Dental Academy (SDA): www.swissdentalacademy.com



More about GBT



Ems-dental.com



EMS 
MAKE ME SMILE.

Reserve a Live GBT Demo: gbt-dental.com/GBT-LIVE-DEMO-ENG